



Laryngectomy Products

SHEFFMED[™]
from Yorkshire *with love*

contents

What is a Laryngectomy? 4

The PHONIQ® System 5

Introduction 5

PHONIQ® DigiClose™ HME Filter 6

PHONIQ® Low Profile HME Filter 8

PHONIQ® Adhesive Baseplate 9

SpiroTect™ 10

SpiroTect™ Baseplates 10

SpiroTect™ Baseplate Wipes 12

SpiroTect™ Protectors 12

Laryngofoam® 13

Laryngofoam Foam Filters 13

Buchanan® 14

Buchanan® Protectors 14

Buchanan® Lite Protectors 16

Miscellaneous 18

Buchanan® Shower Shield 18

Voice Prosthesis Cleaning Brush 18

Office: 0114 261 7161
sales@sheffmed.com

Certified Quality Products

Our products meet the requirements of the most demanding applications and comply with all relevant national and international guidelines and standards. They are registered and approved in many countries world-wide.

Product descriptions


Please note that due to continuous product developments it may be necessary to alter, without notice, products within our range. We endeavour to keep all customers informed of product improvements. Purchased products may differ in appearance from the illustrations shown in this brochure.

© Sheffmed Limited
Sheffmed™ is a Registered Trademark

www.sheffmed.com

What is a Laryngectomy?

A Laryngectomy is surgery to remove the voice box or larynx usually to treat cancer. The larynx (voice box) plays an important role in helping us breathe and speak and protects the airway and lungs when swallowing. The diagrams show a side view of a person's anatomy before and after surgery.

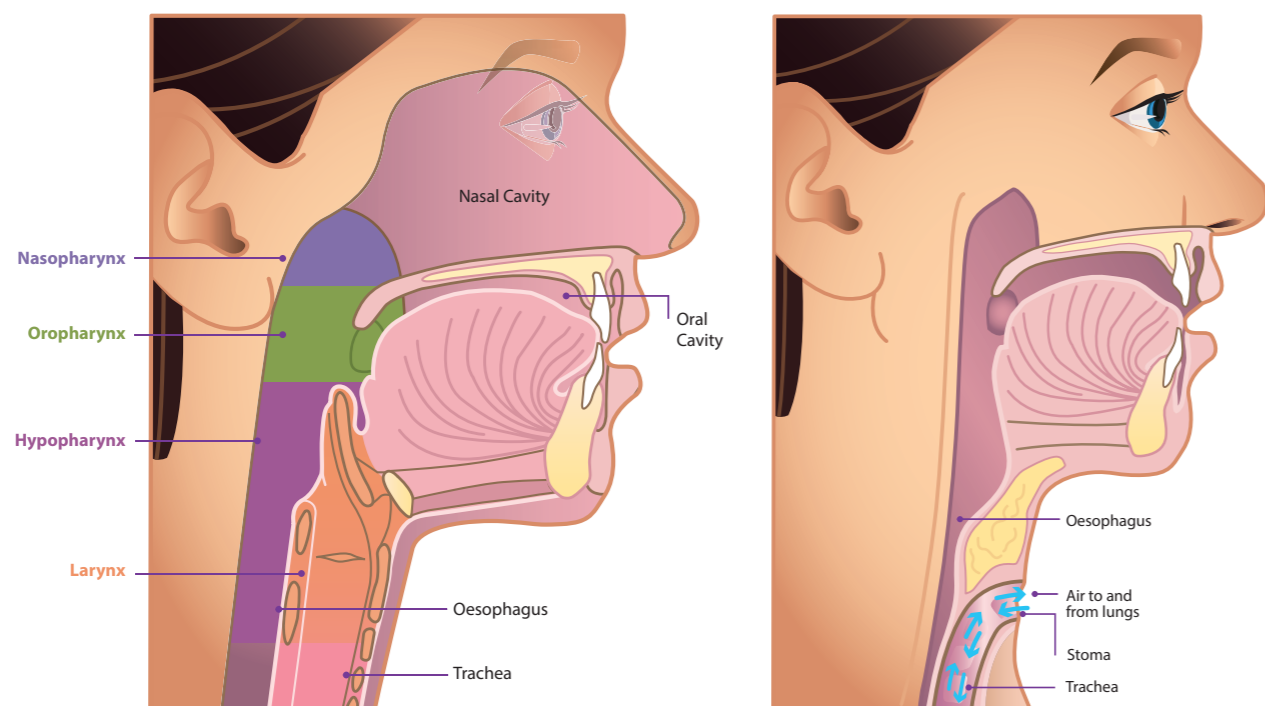

The voice box / larynx (shaded orange in the diagram) is removed and the windpipe / trachea (shaded pink) is brought forward to create an opening on the lower neck called a stoma. Air now travels to and from the lungs directly through the new stoma. The food pipe / oesophagus is surgically repaired so that it is now separate from the trachea.

Considerations after a Laryngectomy:

Prior to undergoing a Laryngectomy a person breathes through the mouth and nose. Air is automatically warmed, filtered and moistened in the nasal cavity prior to reaching the lungs.

After a laryngectomy the mouth and nose are completely separated from the lungs and consequently inhaled air is no longer naturally warmed, filtered or moistened. It is important that the stoma and lungs are protected with a filter system whether the person is active, socialising, showering, resting or sleeping.

A new generation of intelligent laryngectomy products from Sheffmed



The PHONIQ® range is a new generation of Laryngectomy products from Sheffmed with intelligent, patented features which will make meaningful differences to the lives of patients each day.

The PHONIQ® System

Offers comfortable discreet airway filtration and humidification with an interchangeable system of interfaces, HME filter cassettes and shower protection.


PHONIQ® interfaces include two adhesive baseplates both available in round and oval shapes. All PHONIQ® interfaces are compatible with our DigiClose™ HME filters which have a patented cough release capability, and our discreet low profile HME filter.



Featuring a patented **unique pressure release** mechanism

PHONIQ® DigiClose™ HME Filters with Cough Release

PHONIQ® DigiClose™ HME Filters have been carefully designed with intelligent features which improve both usability and aesthetic appearance. PHONIQ® DigiClose™ HME Filters can be worn for 24 hours.

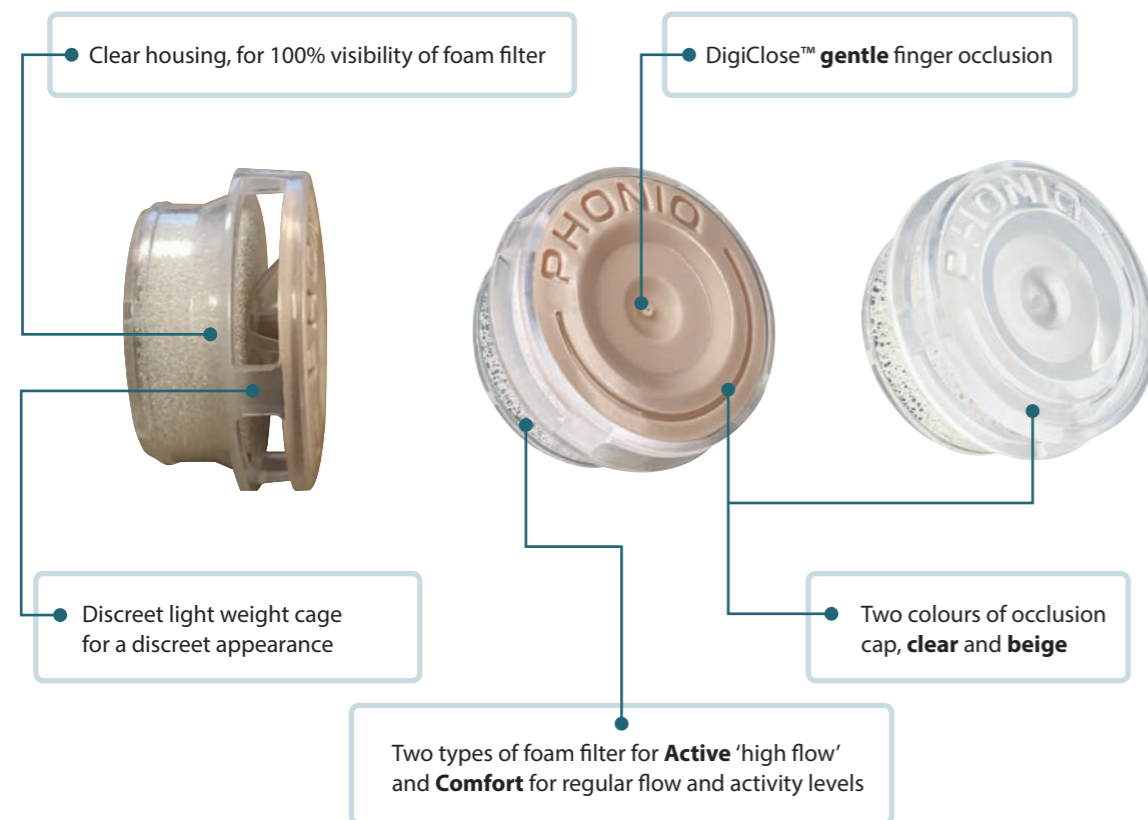

Two Foam Types

PHONIQ® DigiClose™ **ACTIVE** provides low breathing resistance and allows for high flow rates for exercise or possibly those with lung disease. PHONIQ® DigiClose™ **Comfort** allows regular flow and resistance for normal every day activity levels.

Aesthetically Discrete

PHONIQ® DigiClose™ HMEs are lightweight, low profile and available in two colours **beige and clear**.

Enhanced Visibility of Filter

Due to the clear material of the filter housing there is improved visibility of the foam filter, to allow for more timely changes if secretions are present.

PHONIQ® DigiClose™ HME Filters feature a patented unique pressure release mechanism which activates during reflexive coughing or sneezing. This feature helps prevent the HME from being ejected from the baseplate.


1] In its normal operating position the DigiClose™ cap of the HME is flush with its clear housing.



2] During coughing or sneezing the DigiClose™ cap moves quickly forward to allow extra airflow through the filter.



3] The DigiClose™ cap is pressed gently back into place to resume normal functionality.

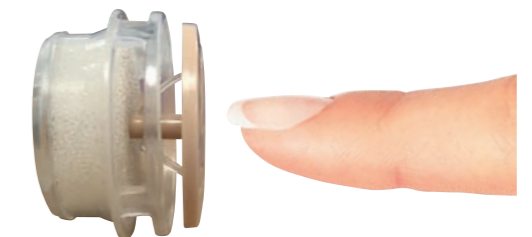

PHONIQ® DigiClose™ HME Filters can be used with PHONIQ® baseplates, laryngectomy tubes and stoma buttons.

PHONIQ® DigiClose™ HME Filters

All Available on NHS Prescription

Ordering information

Ordering information				Airflow and Resistance
LAPFT0001	PhonIQ DigiClose Comfort HME Filter	Clear	Box 30	31.3 mg/L, 0.7cm/H2O under 30l/pm
LAPFT0002	PhonIQ DigiClose Active HME Filter	Clear	Box 30	33.9 mg/L, 0.4cm/H2O under 30l/pm
LAPFT0003	PhonIQ DigiClose Comfort HME Filter	Beige	Box 30	31.3 mg/L, 0.7cm/H2O under 30l/pm
LAPFT0004	PhonIQ DigiClose Active HME Filter	Beige	Box 30	33.9 mg/L, 0.4cm/H2O under 30l/pm

Lower the profile, **not the performance**

PHONIQ® Low Profile

PHONIQ® Low Profile HME Filter

Low Profile

The slim design will make the HME filter more discreet when being worn. The filter fits securely into the interface ring, with none of the filter protruding over the top.

Enhanced Visibility of Filter

Due to the clear material of the filter housing, there is improved visibility of the foam filter, to allow for more timely changes if secretions are present.

Compatibility

The PHONIQ® Low Profile HME Filter is compatible with all PHONIQ® baseplates, and also with laryngectomy tubes and stoma buttons with a 22mm connector.

Excellent HME Capabilities

Despite being half the size of a normal HME filter that's on the market, it still offers high levels of HME capability.



All Available on NHS Prescription

PHONIQ® Low Profile HME Filter

Ordering information

LAPFT0005	PhonIQ Low Profile HME Filter	Box of 30	Airflow and Resistance
			36.3 mg/L, 0.8cm/H2O under 30l/pm

Airflow and Resistance

Designed to promote optimum lung health

PHONIQ® Adhesive Baseplate

PHONIQ® Baseplates adhere gently to the skin area around a tracheostoma to provide an airtight interface between the stoma and PHONIQ® DigiClose™ HME Filter. When used together inhaled air is warmed, moistened and filtered to promote optimum lung health.

PHONIQ® Secure Baseplate

Two Shapes

- Oval for larger surface area
- Round for smaller surface adhesion

Clear flexible material

- Breathable and skin friendly
- Adheres to irregular surfaces
- Suits all skin tones

Secure Adhesive Layer

- Ensures effective seal when worn with an HME




PHONIQ® Hydrocolloid Baseplate

Two Shapes

- Oval for larger surface area
- Round for smaller surface adhesion

Hydrocolloid material

- For sensitive skin
- Thin and soft for improved contouring

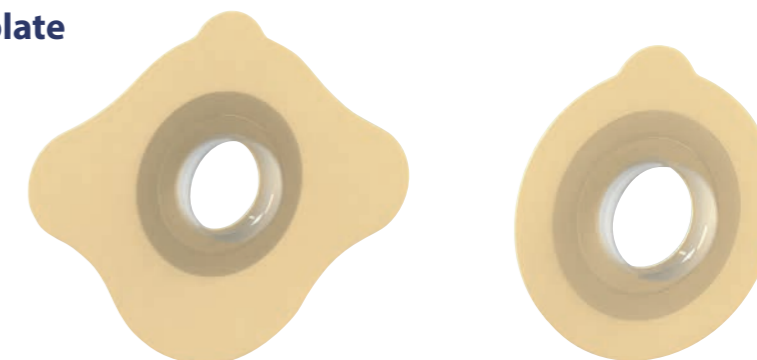

PHONIQ® Baseplates can be worn both DAY and NIGHT for up to 3 days.

PHONIQ® Adhesive Baseplate

Ordering information

LAPBP0001	PhonIQ Secure Baseplate	Oval	Box 20
LAPBP0002	PhonIQ Secure Baseplate	Round	Box 20
LAPBP0003	PhonIQ Hydrocolloid Baseplate	Oval	Box 20
LAPBP0004	PhonIQ Hydrocolloid Baseplate	Round	Box 20

All Available on NHS Prescription

Laryngectomy products with patients in mind

SpiroTect™

All Available
on NHS
Prescription

SpiroTect™ Laryngectomy Products have been designed with the patient in mind. Our extensive range has a variety of prescription products developed to suit the individual needs of Laryngectomees. The variations in baseplates, HMEs and valves mean that we can meet challenges faced by most Laryngectomees. Our SpiroTect™ accessories include adhesive remover wipes and barrier wipes.

SpiroTect™ Latex Free Baseplates

All SpiroTect™ adhesive baseplates have been developed to secure HME Cassettes at the Laryngectomy stoma site. Designed to mould to body contours, these baseplates are available in both round and oval shapes. They are designed to prevent air leakage and provide security, enabling the Laryngectomee to lead a comfortable and quality life.




SpiroTect™ Clear and Strong Baseplate, Oval/Round

The SpiroTect™ Clear and Strong Baseplate Oval and Round are skin friendly baseplates that secure HME Cassettes at the Laryngectomy stoma site.

Ordering information

PTW300	Clear and Strong Baseplate - Oval	Pack of 20
PTW301	Clear and Strong Baseplate - Round	Pack of 20

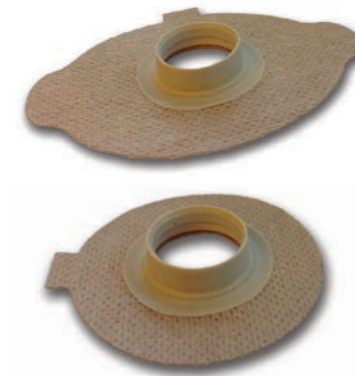

SpiroTect™ Fabric Baseplate, Oval/Round

The SpiroTect™ Fabric Baseplate Oval and Round are skin friendly fabric dressings that secure HME Cassettes at the Laryngectomy stoma site.

Ordering information

PTW100	Fabric - Oval	Pack of 20
PTW101	Fabric - Round	Pack of 20



SpiroTect™ Sensitive Baseplate, Oval/Round

The SpiroTect™ Sensitive Baseplate Oval and Round are skin friendly dressings made from hydrocolloid that secure HME Cassettes at the Laryngectomy stoma site. They can be used immediately post operatively and for those with sensitive skin.

Ordering information

PTW200	Sensitive - Oval	Pack of 20
PTW201	Sensitive - Round	Pack of 20



SpiroTect™ Secure Convex Baseplate

The SpiroTect™ Secure Convex Baseplate is a skin friendly baseplate that has been developed for a more precise, supportive and secure positioning around the stoma site. Ideal for Laryngectomees with deep tracheostoma who find air leakage with their current baseplate is preventing them from regularly speaking.

Ordering information

PTW700	Secure Convex	Pack of 20
--------	---------------	------------



SpiroTect™ Baseplate Barrier Wipe

Available on Prescription

The SpiroTect™ Baseplate Barrier Wipe should be used before the application of a baseplate to provide a protective layer for the skin to reduce irritation and/or damage from mucus or adhesives.

Ordering information

PTA500 SpiroTect™ Baseplate Barrier Wipe Pack of 30



SpiroTect™ Baseplate Adhesive Remover Wipe

Available on Prescription

The SpiroTect™ Baseplate Adhesive Remover Wipe breaks down adhesive from baseplates and provides a fast and effective removal of sticky residue.

Ordering information

PTA600 SpiroTect™ Baseplate Adhesive Remover Wipe Pack of 30

All Available
on NHS
Prescription

SpiroTect™ Protector

Available in two sizes, SpiroTect™ Protectors are specifically designed to safeguard the airways of Laryngectomy and Tracheostomy patients, giving them the confidence to perform everyday tasks.

Available
on NHS
Prescription

- Available on prescription
- Filters, warms and moistens air to help protect the trachea
- Ezi-Tie® adjusters ensure simple and effective fastening.
- No label on the Protector to avoid discomfort
- Protector size: Large 255mm x 195mm, Small 150mm x 120mm

* Protector should be hand washed no more than three times

SpiroTect™ Protector

Ordering information

PTA700	SpiroTect™ Protector - Small, Navy	Pack of 10
PTA702	SpiroTect™ Protector - Large, Navy	Pack of 10



Lightweight, comfortable and unobtrusive protection featuring the unique Hydrolox® 1 Foam

Laryngofoam


Disposable Foam Filters

Key Features

- Soft, rounded edges
- The only filters with Hydrolox® 1 Foam
- Available on FP10 Prescription

The Laryngofoam Filter warms, moistens and filters the incoming air and can be applied in seconds.

- Laryngofoam filters are held securely in place by a hypoallergenic adhesive strip
- The simplicity and effectiveness of this product ensures its popularity throughout the world with laryngectomees in all climates and conditions
- Laryngofoam has excellent moisture transfer properties which means better humidification within the trachea for breathing comfort and confidence
- Soft rounded edges are a unique feature of the Laryngofoam range

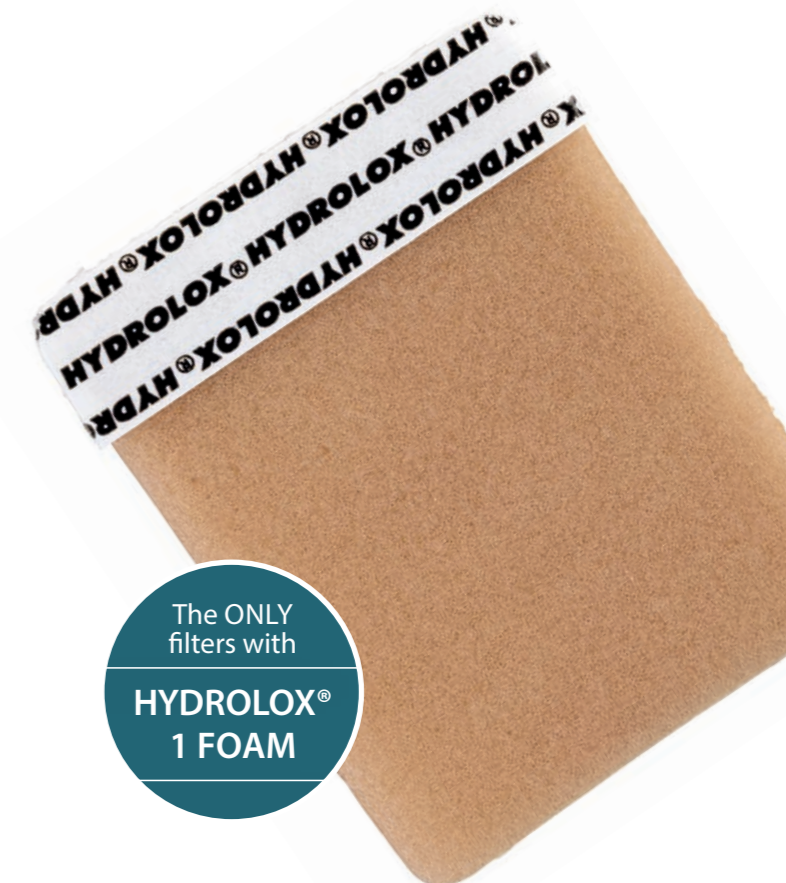

- Laryngofoam foam filters are supplied in packs of 30. Use 1-2 per day as necessary
- The only filters with Hydrolox® 1 Foam

Laryngofoam Foam Filters

Ordering information

LALFF0003	Beige	Small	65 x 51 x 4 mm	Pack of 30
LALFF0023	Beige	Large	65 x 65 x 4 mm	Pack of 30

Available
on FP10
Prescription



Scientifically **proven** protection, filtration and humidification

Buchanan®

Laryngectomy and Tracheostomy Protector

Key Features

- Scientifically proven benefits
- Soft and comfortable
- Effective filtration and humidification
- Can be worn day and night
- Available in two sizes

The Buchanan® Protector is a neck worn cover that provides filtration and delivers efficient heat and moisture exchange.

Buchanan delivers unique benefits to neck breathers through effective product design and material composition.

The crucial benefits are delivered by the exclusive Hydrolox® 1 Foam especially designed for effective heat and moisture delivery.

Carefully selected materials such as pure cotton mesh and the special Hydrolox 1 Foam provide the following benefits:

- Comfortable and soft to the skin with no sharp edges
- Non-adhesive therefore no skin irritation
- Flexibility, so rests comfortably on the neck and chest
- Can be used immediately, post-operatively
- Ideal in all weather conditions
- Choice of sizes - small and large
- Can be worn day and night
- Can be hand washed in max 30°C up to 3 times

Buchanan is the only airway protector with Hydrolox® 1 Foam offering the following benefits:

- Excellent moisture absorbing capacity
- Provides heat and moisture to incoming air
- Filters out dust and dirt

Buchanan Large

Dimensions: Width: 220 mm Length: 190 mm

Ordering information

LABUP0001	Large	Pack of 10
-----------	-------	------------

Buchanan Small

Dimensions: Width: 145 mm Length: 120 mm

Ordering information

LABUP0002	Small	Pack of 10
-----------	-------	------------


The Buchanan® Range eliminates up to 97% of airborne particles equivalent to the size of mould spores, dust and some bacteria, which are small enough to be inhaled and reach the middle of the lungs.



Ezi-Tie® adjusters for increased comfort

The **only** protectors with **HYDROLOX® 1** FOAM

Available on FP10 Prescription

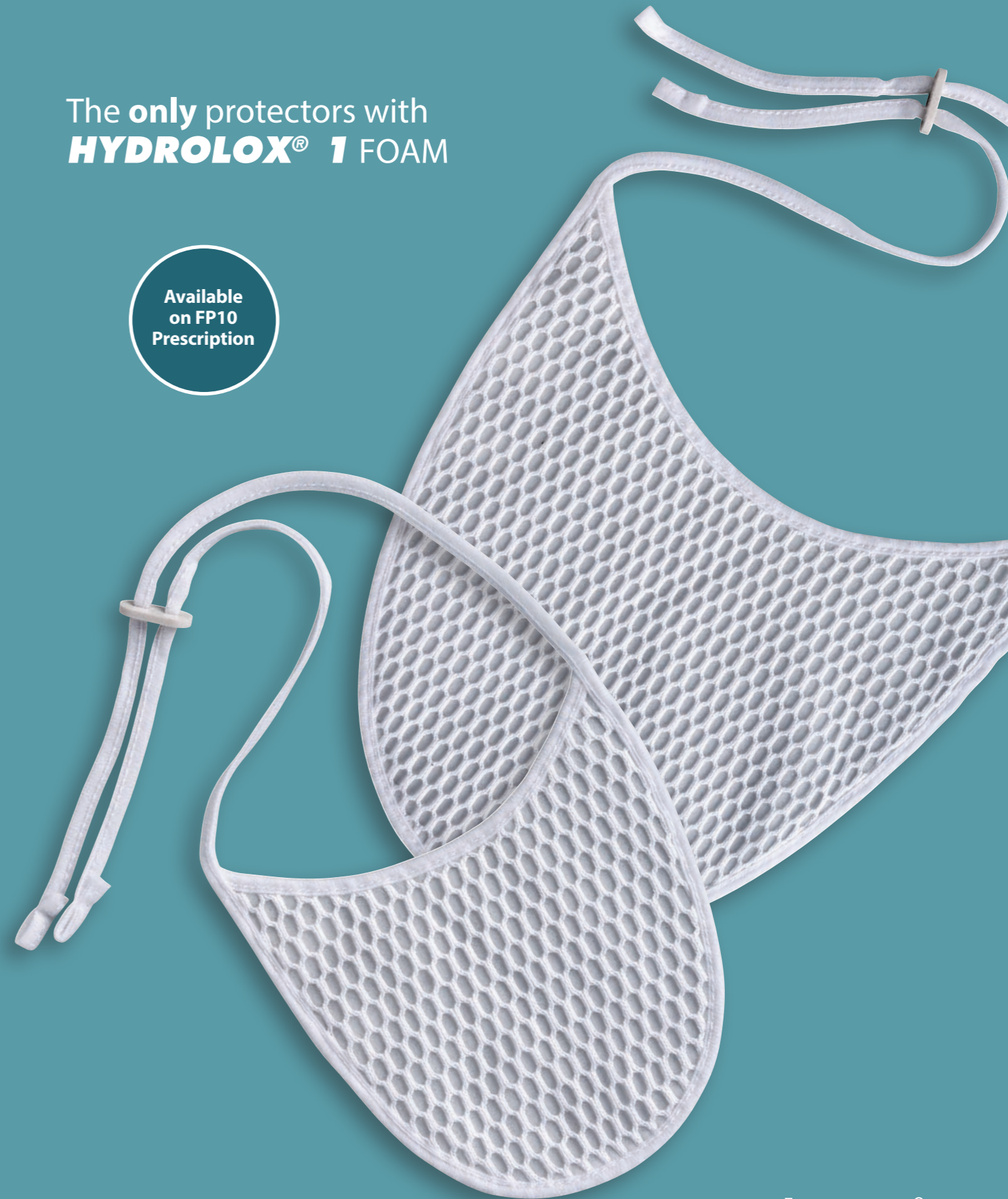

Buchanan®

Better breathing, better stoma care

Scientifically **proven** protection, filtration and humidification

Buchanan® Lite

Laryngectomy and Tracheostomy Protector

Key Features

- Lightweight and comfortable
- High performance
- Effective filtration and humidification
- Scientifically proven benefits
- Choice of colours

Buchanan® Lite Large

Dimensions: Width: 225 mm Length: 195 mm

Ordering information

LABLT1001	Large	White	Pack of 10
LABLT1002	Large	Blue	Pack of 10
LABLT1003	Large	Beige	Pack of 10

Buchanan® Lite Small

Dimensions: Width: 150 mm Length: 120 mm

Ordering information


LABLT2001	Small	White	Pack of 10
LABLT2002	Small	Blue	Pack of 10
LABLT2003	Small	Beige	Pack of 10

The Buchanan® Lite Protector offers additional features over the original Buchanan design and offers a wider choice of colours.


- Essential fine filtration through the dual layer microfibre filter layer
- Excellent moisture retention
- The breathable polyester non-woven design makes the product lightweight and soft
- Ezi-Tie® adjusters ensure simple and effective fastening
- Can be hand washed in max 30°C up to 3 times

The Buchanan® Range eliminates up to 97% of airborne particles equivalent to the size of mould spores, dust and some bacteria, which are small enough to be inhaled and reach the middle of the lungs.

Ezi-Tie® adjusters for increased comfort

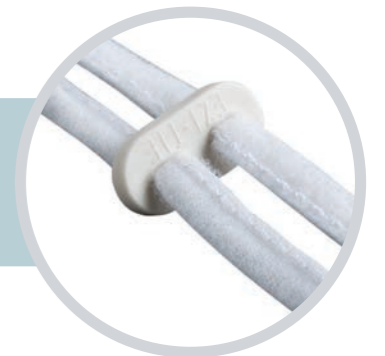

The unique layered lightweight construction

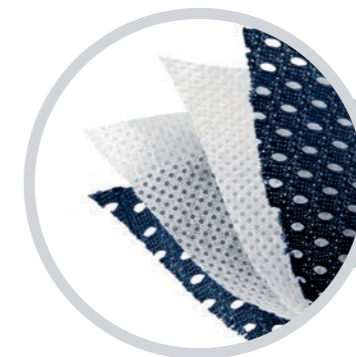

Each colour available in two sizes

COMING SOON!
The Black Buchanan® Lite Range

Available on FP10 Prescription

Buchanan® Lite
Essential airway protection and comfort

Management Aids

Buchanan® Shower Shield

The PVC material helps prevent water entering the stoma as it is waterproof. For breathing comfort we advise that a protector or filter be worn underneath the Shower Shield. Please ensure that the Shower Shield fits snugly against the neckline to form a seal. Note: For deep stomas extra care is needed if using this product. Colours may vary from time to time.

Buchanan® Shower Shield

Ordering information

LASHC0001 Buchanan® Shower Shield Waterproof Cover - Blue



Voiceline Voice Prosthesis Cleaning Brush

Featuring an adjustable safety stop (registered design), which enables you to place the brush into your prosthesis, it reduces the risk of the brush passing through the prosthesis causing injury.

Voiceline Voice Prosthesis Cleaning Brush

Ordering information

LAPCB0003 Pack of 10



Available on FP10 Prescription

Sheffmed Partners with The Swallows Charity to Support Laryngectomy Patients

Sheffield-based medical supplier **Sheffmed** has announced a new partnership with **The Swallows Head and Neck Cancer Support Group**, strengthening its commitment to improving care and support for patients and families affected by head and neck cancer.

As part of the collaboration, Sheffmed will donate essential products for inclusion in **The Swallows' Support Boxes**, which are distributed to newly diagnosed patients and those recovering after treatment.

The company is also providing sponsorship for key initiatives, including:

- **The Swallows' 24/7 Text Support Line**, offering round-the-clock advice and reassurance to patients and caregivers. 07830 929 400
- **Emergency Window Stickers**, which provide critical identification details to emergency services and are distributed nationwide.

Sheffmed has also been added to the charity's official list of "**Friends of The Swallows**", highlighting its role as a committed partner in supporting people affected by head and neck cancer.

Chris Curtis, CEO of The Swallows Charity, welcomed the partnership:

"Partnerships like this with Sheffmed are vital. They allow us to reach more patients at the very moment they need help, whether it's through a support box, a reassuring phone call, or life-saving emergency information. Together, we can ensure no patient has to face their journey alone."

About The Swallows

The Swallows Head and Neck Cancer Support Group is a UK-based charity supporting patients, caregivers, and families affected by head and neck cancers. With a global reach, the charity provides practical and emotional support through services such as a 24/7 text and helpline, peer-to-peer connections, and essential resources for patients undergoing treatment and recovery. Its mission is to ensure that no patient or caregiver ever faces their journey alone.

www.theswallows.org.uk



Sheffmed are proud to support NALC

NALC look after the interests of laryngectomees - people who have had a laryngectomy - and their families and friends. They offer a vital forum for laryngectomees through their 60 clubs up and down the country.

NALC also support a wide range of healthcare professionals in the UK and worldwide, encouraging education, collaboration and research.

www.laryngectomy.org.uk





Sheffmed Ltd

Unit 4, Coggin Mill Way,
Centurion Business Park,
Bessemer Way,
Rotherham S60 1FB

Tel: 0114 261 7161

Email: sales@sheffmed.com

www.sheffmed.com



LIT3/0111 REV_06_01_26



SHEFFMED™
from Yorkshire *with love*